


Instructions for Use, eZinject Catheter

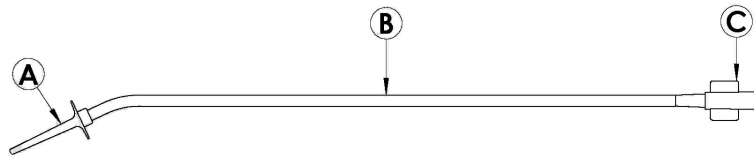

eZinject catheter

- A. Cervical Stop
- B. Shaft
- C. Luer

DESCRIPTION:

The eZinject catheter is a uterine injector for single use. This is a sterile disposable device made of polyvinyl chloride, which meets USP recommendations for implant testing. The product is designed with a single lumen for injection of fluid through a distal tip. The product is slightly curved to facilitate easy entrance to the cervical canal. The distal tip is a tapered design with a cervical stop, preventing excessive penetration. The proximal end is equipped with a luer fitting on a flexible sleeve to accommodate a syringe.

INDICATIONS:

The eZinject catheter is indicated for such procedures as Sonohysterography. It is designed for “in office use.” This device can be applied without cervical dilation or anesthesia if the uterus can be sounded easily with a standard uterine sound. The eZinject catheter provides for efficient sealing of the cervix when injections of liquid are required.

CONTRAINDICATIONS:

- Pregnancy
- Uterine or tubal infection

WARNINGS:

- To avoid uterine trauma, insert the eZinject catheter along the natural axis of the cervical canal.

PRECAUTIONS:

- Lubricate distal end for easy insertion.
- Observe manufacturer’s directions and precautions pertaining to any liquid media or gas that is being injected into the patient.

ADVERSE REACTIONS:

- Cramping
- Infection

DIRECTIONS FOR USE:

1. Check to see that the sterile pouch containing the eZinject catheter has not been punctured or damaged thus compromising sterility.
2. Using a speculum, expose the cervix and grasp the anterior lip with a single tooth tenaculum.
3. Lubricate the distal tip for easy insertion. Carefully guide the instrument along the natural axis of the cervix to avoid injury.
4. Injected media may be applied with a standard syringe or extenders as needed.
5. Carefully remove the eZinject catheter after all injecting is completed.