## INTRAUTERINE INSEMINATION (IUI) CATHETER

### DESCRIPTION:

Intrauterine Insemination (IUI) catheter is a single use, disposable catheter used primarily to inject sperm into the uterus. The distal end is formed into an atraumatic shape for smooth insertion and has a side port through which fluid exits the catheter. Side hole at tip is aligned with the printed depth marks. The proximal end has a luer connector for attaching the syringe with the fluid for injection. The catheter shaft has 1cm depth marks to place catheter to desired uterine depth. Clinician can align catheter to inject toward the left or right fallopian tube ostium.

The Shapeable IUI catheter is malleable to aid catheter placement.

### INDICATIONS:

Delivery of sperm into the uterus

### PRECAUTIONS:

Do not use if expiration date has passed

### WARNINGS:

This device is only for use for patients designated for intrauterine insemination.

### DIRECTION FOR USE:

1. Prepare patient and sperm according to standard clinical practice.
2. Remove catheter from sterile packaging.
3. Sound uterus to determine uterine depth.
4. Connect insemination syringe to luer connector of catheter.
5. For Shapeable IUI, bend shaft as desired to aid catheter placement.
6. Using depth marks on catheter, insert catheter through cervical canal to sounded depth.
7. Inject sperm according to clinical procedure.
8. Carefully withdraw and discard catheter and syringe.

### MANUFACTURED BY:

CRI  
dba Thomas Medical, Inc  
6102 Victory Way  
Indianapolis, IN 46278 USA  
Telephone: (800) 556-0349  
(317) 872-0074  
Facsimile: (317) 872-0169

Emergo Europe  
Molenstraat 15  
2513 BH The Hague  
The Netherlands  
T: 31(0) 70.345.8570  
F: 31(0) 70.346.7299

Comments regarding this device can be directed to  
Attn: Quality Assurance Dept., CRI, 6102 Victory Way,  
Indianapolis, IN 46278 USA

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