## DIRECTIONS FOR USE

FLEXIBLE CERVICAL DILATOR™  
Product # OS-01-001

Figure 1

### FLEXIBLE CERVICAL DILATOR™

- 1) Narrow End

This instrument is used to find and dilate the cervical os and canals prior to use of the Zinnanti Sampler™ (endometrial suction curette).

**CAUTION:** Federal Law (USA) restricts the sale of this device by or on the order of a physician only.

### PRECAUTIONS:

- Contents sterile in unopened, undamaged package.
- Single Patient Use Only – Do not Resterilize.
- Do not use a damaged TMI Flexible Cervical Dilator™

### INDICATION AND USAGE

This instrument is especially useful in finding and dilating the cervical os and cervical canal in the postmenopausal patient with cervical stenosis.

### CONTRAINDICATIONS:

This instrument should not be used when the following conditions exist:

- Pregnancy or suspicion of pregnancy.
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum (cervicitis, endometritis, tubo-ovarian inflammatory diseases, or pelvic inflammatory disease).
- Known or suspected cervical changes secondary to diethylstilbestrol (DES) intrauterine exposure.
- Invasive cancer that is visible on examination.

### DIRECTIONS FOR USE:

The Flexible Cervical Dilator™ is used to find the external os by simply pushing the narrow end gently against the cervix until it finds its way. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal os.

### PRODUCT INFORMATION:

Product Number: OS-01-001  
OD Size: 2.0mm – 4.0mm taper  
Length: 21.5cm  
Packaging: 25 or 50 sterile units per box

MANUFACTURED BY:  
Catheter Research, Inc.  
dba Thomas Medical, Inc.  
6102 Victory Way  
Indianapolis, IN 46278  
Telephone: (800) 556-0349  
(317) 872-0074  
Facsimile: (317) 872-0169

AUTHORIZED REPRESENTATIVE:  
Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Comments regarding this device can be directed to Attn: Quality Assurance Dept., CRI, 6102 Victory Way, Indianapolis, IN 46278 USA, Fax: (317) 872-0169

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FLEXIBLE CERVICAL DILATOR™ (FLEXIBELE  
CERVICALE DILATATOR)  
Product # OS-01-001

Figure 1

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