**DIRECTIONS FOR USE**  
**DISPOSABLE UTERINE SOUND**  
 Product # US-01-001P

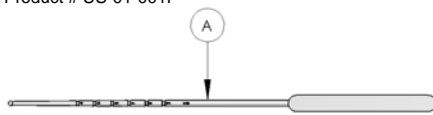
Figure 1

**Printed Disposable Uterine Sound**

A. Shaft

This instrument is used to dilate the cervical os and canal while taking a sounding of the uterus. Using the centimeter markings on the shaft of the instrument the user will know how deep the sounding depth is. The flexibility of the instrument allows the user to know what direction the uterus is positioned in.

**CAUTION:** Federal Law (USA) restricts the sale of this device by or on the order of a physician only.

**PRECAUTIONS:**

- Contents sterile in unopened, undamaged package.
- Single Patient Use Only – Do not Resterilize.
- Do not use a damaged TMI Uterine Sound

**INDICATION AND USAGE**

This instrument is especially useful in dilating the cervical os and cervical canal while taking a sounding measurement of the uterus.

**CONTRAINDICATIONS:**

This instrument should not be used when the following conditions exist:

- Pregnancy or suspicion of pregnancy.
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum (cervicitis, endometritis, tubo-ovarian inflammatory diseases, or pelvic inflammatory disease).
- Known or suspected cervical changes secondary to diethylstilbestrol (DES) intrauterine exposure.
- Invasive cancer that is visible on examination.

**DIRECTIONS FOR USE:**

Gently insert (lubricating the shaft) the Uterine Sound into the cervical os. Move the instrument forward until resistance is felt at the fundus. At this point a sounding depth can be read.

**PRODUCT INFORMATION:**

Product Number: US-01-001P  
 OD Size: 3.5mm  
 Length: 24.8cm  
 Packaging: 25 sterile units per box

**MANUFACTURED BY:**

**CRI**  
 dba Thomas Medical, Inc.  
 6102 Victory Way  
 Indianapolis, IN 46278  
 Telephone: (800) 556-0349  
 (317) 872-0074  
 Facsimile: (317) 872-0169

**AUTHORIZED REPRESENTATIVE:**

Emergo Europe  
 Prinsessegracht 20  
 2514 AP The Hague  
 The Netherlands

Comments regarding this device can be directed to Attn: Quality Assurance Dept., CRI, 6102 Victory Way, Indianapolis, IN 46278 USA, Fax: (317) 872-0169

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dbi Thomas Medical, Inc.