

HSG Tray

for

Sonohysterography and Hysterosalpingography

CONTENTS:

1 EA	EXTENSION TUBING, .050"ID x 36" M/F LL	1 EA	PAD, MAXI (WRAPPED)
1 EA	SYRINGE, 20CC L/L	3 EA	SWAB, COTTON 8"
2 EA	PVP SOLUTION 1oz.	1 EA	BAG, HEADER 14"x18"
1 PK	JELLY, LUBRICATING 3g	1 EA	DISPOSABLE VAGINAL SPECULUM.
1 EA	OINTMENT, PVP 1g	1 EA	WRAP, POLY 30"x30"
1 EA	DRAPE, WITH 2 ½" FENESTRATION, 18" x 26"	1 EA	TRAY, 16"x6.6"x2.3"
1 EA	NEEDLE, 18G x 1 ½"	1 EA	TRAY, 5.3"x4.5"x2"
5 EA	GAUZE, 3"x3" 4PLY	1 EA	CERVICAL DILATOR (GREEN)
2 EA	STERILE GAUZE, 3x3 12PLY	2 EA	SPONGE STICK, 6"

STERILE UNLESS INTEGRITY OF PACKAGE IS COMPROMISED. SINGLE USE ONLY. MADE IN THE U.S.A. STERILIZED WITH ETO. FOLLOW RECOMMENDED HEALTHCARE PROCEDURES. CAUTION: FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED PRACTITIONER. STORE IN A COOL, DRY AREA. AVOID FREEZING OR EXCESSIVE HEAT.

LOT NUMBER: XXXX

EXPIRY DATE: YYYY-MM-DD

STERILE EO



NOTES:

1. The Drape with fenestration is used to create a sterile field.
2. The 30" x 30" poly wrap may be placed under the patient, cotton side up, to be used as an absorbent pad.
3. The PVP Solution and cotton swabs are used to prepare the vaginal area.
4. Lubricating jelly is provided to assist cervical dilator or catheter insertion.
5. The cervical dilator is used to find the external os by pushing the narrow end gently against the cervix until it enters the cervical canal. The shape retaining tip may be curved as necessary to facilitate insertion. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal os.
5. The 20cc syringe and 18G needle are provided to withdraw contrast media from its container.
6. The 36" extension tubing can be connected to the blue hub of the HS catheter to allow injection of contrast media from outside the x-ray field during Hysterosalpingography procedures.

This product is not made with natural rubber latex.