100 W. Industrial Rd. Addison, IL 60101 USA

toll-free: 800.556.0349

sales@thomasmedical.com

FLEXIBLE CERVICAL DILATOR

Flexible Cervical Dilator

1. Narrow End

CAUTION:

Federal Law (USA) restricts the sale of this device by or on the order of a physician only.

PRECAUTIONS:

- Contents sterile in unopened, undamaged package.
- Single Patient Use Only – Do not Resterilize.
- Do not use a damaged TMI Flexible Cervical Dilator.

INDICATION AND USAGE:

This instrument is especially useful in finding and dilating the cervical os and cervical canal in the postmenopausal patient with cervical stenosis.

CONTRAINDICATIONS:

This instrument should not be used when the following conditions exist:

- Pregnancy or suspicion of pregnancy.
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum (cervicitis, endometritis, tubo-ovarian inflammatory diseases, or pelvic inflammatory disease).
- Known or suspected cervical changes secondary to diethylstilbestrol (DES) intrauterine exposure.
- Invasive cancer that is visible on examination.

DIRECTIONS FOR USE:

The Flexible Cervical Dilator is used to find the external os by simply pushing the narrow end gently against the cervix until it finds its way. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal os.

PRODUCT INFORMATION:

Product Number: OS-01-001
OD Size: 2.0mm – 4.0mm taper
Length: 21.5cm
Packaging: 50 sterile units per box

MANUFACTURED BY:
Thomas Medical, Inc.
100 W. Industrial Rd. Addison,
IL 60101 USA
Telephone: (800) 556-0349
Fax: (888) 411-3754

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