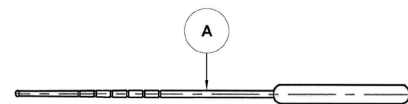
100 W. Industrial Rd. Addison, IL 60101 USA

toll-free: 800.556.0349

sales@thomasmedical.com

DISPOSABLE UTERINE SOUND

Disposable Uterine Sound™

A. Shaft

This instrument is used to dilate the cervical os and canal while taking a sounding of the uterus. Using the centimeter markings on the shaft of the instrument the user will know how deep the sounding depth. The flexibility of the instrument allows the user to know what direction the uterus is positioned in.

CAUTION:

Federal Law (USA) restricts the sale of this device by or on the order of a physician only.

PRECAUTIONS:

- Contents sterile in unopened, undamaged package.
- Single Patient Use Only – Do not Resterilize.
- Do not use a damaged TMI Flexible Cervical Dilator, Uterine Sound

INDICATION AND USAGE:

This instrument is especially useful in dilating the cervical os and cervical canal while taking a sounding measurement of the uterus.

CONTRAINDICATIONS:

This instrument should not be used when the following conditions exist:

- Pregnancy or suspicion of pregnancy.
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum (cervicitis, endometritis, tubo-ovarian inflammatory diseases, or pelvic inflammatory disease).
- Known or suspected cervical changes secondary to diethylstilbestrol (DES) intrauterine exposure.
- Invasive cancer that is visible on examination.

DIRECTIONS FOR USE:

Gently insert (lubricating the shaft) the Uterine Sound into the cervical os. Move the instrument forward until resistance is felt at the fundus. At this point a sounding depth can be read.

PRODUCT INFORMATION:

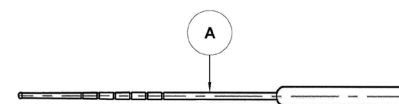
Product Number: US-01-001
OD Size: 3.5mm
Length: 24.8cm
Packaging: 25 sterile units per box

MANUFACTURED BY:
Thomas Medical, Inc.
100 W. Industrial Rd. Addison,
IL 60101 USA
Telephone: (800) 556-0349
Fax: (888) 411-3754

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