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## Symbol Descriptions: English

- In this document, indicates conditions or practices that could lead to illness, injury, or death („WARNING“).
- In this document, indicates conditions or practices that could damage the equipment or other property („CAUTION“).
- Consult Instructions for Use
- Do not re-use, Single use device
- Do not re-sterilize
- Catalogue Number
- Lot Number
- Use By Date
- Manufacture Date
- Quantity
- Manufacturer's contact information
- Not made with natural rubber latex
- Fragile
- Temperature limit
- Humidity limitation
- For Professional Use
- Do not use if package is damaged

**STERILE/O** Sterilized using ethylene oxide (only applicable to versions marked as sterile on unit packaging)

## Indications for Use

The single-use lighted vaginal speculum is indicated for use on women undergoing a procedure requiring vaginal access and exposure (e.g. for pelvic examination or Pap Smear). The device uses opposing bi-valve plastic blades to separate the vaginal wall and is intended to expose the interior of the vagina and exterior of the cervix.

- Warnings and Cautions**  
**WARNING:** Inspect each device for damage prior to use.  
**WARNING:** If a device is damaged, do not use any device from the same box.  
**WARNING:** When in transit or storage, device may be subject to damage beyond the control of the manufacturer or supplier.  
**WARNING:** Never use device with laser equipment.  
**WARNING:** Treat used devices as bio hazardous infectious material. Dispose of used device in suitable disposal unit or in accordance with local regulations.  
**WARNING:** Devices are not compatible with gamma radiation or autoclave sterilization and should never be re-sterilized by any method.  
**WARNING:** No modification of this equipment is allowed.  
**WARNING - Risk of reuse:** The strength of the device may be significantly compromised and the resulting risk to the patient and/or user is unknown.  
**CAUTION:** Use by trained personnel only.  
**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician.  
**CAUTION:** Light source is single-use and of limited duration. Do not activate light source until you are ready to use the device.

**Warranty**  
Please visit [www.obpmedical.com](http://www.obpmedical.com) "Terms and Conditions" page for warranty information. OBP Medical should be notified immediately via phone or email of any defective units.

# obp™

simple, bright ideas.

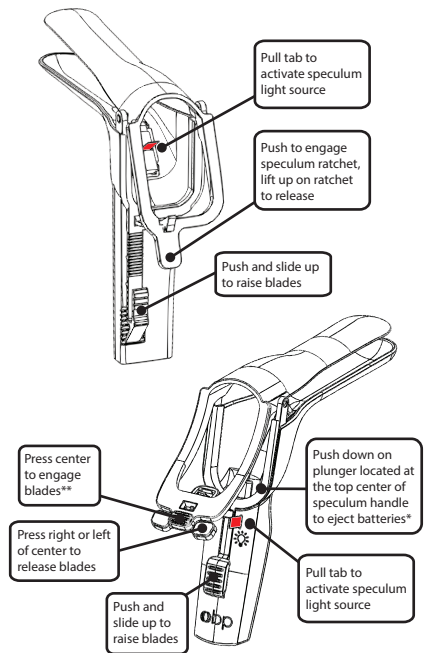
**ER-SPEC®** Single-Use Vaginal Speculum with Integrated LED Light Source

**HER-SPEC™** Single-Use Click-Free Vaginal Speculum with Integrated LED Light Source

**REF C020001 thru C029999**

Includes **OFFICESPEC®** side-opening model

**OBP Medical**  
360 Merrimack Street  
Building 9  
Lawrence, MA 01843 USA  
+ 1-978-291-6853  
[www.obpmedical.com](http://www.obpmedical.com)

\*batteries eject downward through handle \*\*engagement requires counter-force

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