## EMBRYO TRANSFER CATHETER

### DESCRIPTION:

Embryo transfer catheter sets are comprised of a transfer catheter and a guide catheter. The devices consist of a closed inner catheter and a detachable outer sheath that is attached to the inner catheter by a Luer compatible adapter. The inner catheter is 18g with rounded tip. Length of Embryo Transfer Catheter is 18 to 23cm.

### INDICATIONS:

The Embryo Transfer Catheter is used to place in vitro fertilized (IVF) embryos into the uterine cavity.

### CONTRAINDICATIONS:

This device should not be used in the presence of:

- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device
- Uterine perforation

### WARNINGS:

The catheter should never be forced against digital resistance while inside the uterine cavity, as forcing the catheter may result in damage to the endometrial tissue and bleeding. The outer sheath should not be advanced further than the internal, and should certainly never enter the uterine cavity, as this may result in damage to the endometrial tissue and bleeding. The inner catheter should not be pulled back further than the last graduation. Removing it from the outer sheath completely may result in damage to the endometrial tissue and bleeding.

### CAUTION:

Federal law restricts this device to sale by or on the order of a physician.

### PRECAUTIONS:

Infection may occur due to bacterial contamination of the device during vaginal manipulation. This may result in urinary tract infection (UTI), pelvic inflammatory disease (PID), or uterine infection. Bleeding may occur as a result of trauma due to insertion of the catheter through the cervix. Embryo transfer procedures should only be performed by physicians trained in Assisted Reproductive Techniques.

### ADVERSE REACTIONS:

There have not been any reports of any adverse reactions to the Embryo Transfer Catheter.

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