 **800 Pasquinelli Dr. Westmont, IL 60559 USA**

 **toll-free: 800.556.0349**

 **sales@thomasmedical.com**

EMBRYO TRANSFER CATHETER

DESCRIPTION:

Embryo transfer catheter sets are comprised of a transfer catheter and a guide catheter. The devices consist of a closed inner catheter and a detachable outer sheath that is attached to the inner catheter by a Luer compatible adapter. The inner catheter is 18g with rounded tip. Length of Embryo Transfer Catheter is 18 to 23cm.

INDICATIONS:

The Embryo Transfer Catheter is used to place in vitro fertilized (IVF) embryos into the uterine cavity.

CONTRAINDICATIONS:

This device should not be used in the presence of:

- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device
- Uterine perforation

WARNINGS:

The catheter should never be forced against digitally felt resistance while inside the uterine cavity, as forcing the catheter may result in damage to the endometrial tissue and bleeding. The outer sheath should not be advanced further than the internal, and should certainly never enter the uterine cavity, as this may result in damage to the endometrial tissue and bleeding. The inner catheter should not be pulled back further than the last graduation. Removing it from the outer sheath completely may result in damage to the endometrial tissue and bleeding.

CAUTION:

Federal law restricts this device to sale by or on the order of a physician.

PRECAUTIONS:

Infection may occur due to bacterial contamination of the device during vaginal manipulation. This may result in urinary tract infection (UTI), pelvic inflammatory disease (PID), or uterine infection. Bleeding may occur as a result of trauma due to insertion of the catheter through the cervix. Embryo transfer procedures should only be performed by physicians trained in Assisted Reproductive Techniques.

ADVERSE REACTIONS:

There have not been any reports of any adverse reactions to the Embryo Transfer Catheter.

INSTRUCTIONS FOR USE:

1. Place patient in a lithotomy or knee/chest position.
2. Insert vaginal speculum to expose cervix.
3. If applicable, place obturator into guide catheter to aid in traversing the cervix.
4. Insert guide catheter through cervix. NOTE: Some guide catheters have markings or positioners to aid proper placement.
5. If applicable, remove obturator prior to placement of transfer catheter.
6. Attach syringe to luer-style fitting of transfer catheter and follow laboratory procedure for loading embryos into catheter.
7. Insert transfer catheter into guide catheter.
8. Advance transfer catheter until the tip is in desired location within the uterine cavity. NOTE: Ultrasound can be used to locate the tip of the transfer catheter. Some catheters are equipped with bands for enhanced visualization under ultrasound.
9. Expel embryos into the uterine cavity.
10. Remove catheters and check for embryo retention.

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