 **800 Pasquinelli Dr. Westmont, IL 60559 USA**

 **toll-free: 800.556.0349**

 **sales@thomasmedical.com**

HSG PROCEDURE TRAY

DEVICE DESCRIPTION

The H/S Procedure Tray contains components routinely used for conducting Hysterosonography or Hysterosalpingography procedures. The H/S Catheter Set provided in the tray is not compatible with oil-based contrast media. This product is not made with natural rubber latex.

The H/S Procedure Tray contains one sanitary napkin, one lubricant jelly, two packets (¾ oz) povidone iodine solutions, two 6" sponge swabs, one fenestrated drape, five gauze pads 3" x 3", 2 each sterile gauze 3"x3" 12ply, one over-wrap 30" x 30", one 18 gauge needle, three 8" swabs, one 20 - 30ml syringe, one 36" extension tube set, one disposable vaginal speculum, one cervical dilator. Options include Flexible or Shapeable HSG Catheter (5F or 7F) or tray without catheter. All tray components are sterile for single use only.

INTENDED USE/INDICATIONS:

The H/S Procedure Tray components are used to administer contrast media during Hysterosonography or Hysterosalpingography procedures.

CONTRAINDICATIONS:

Hysterosonography or Hysterosalpingography should not be performed under the following conditions: known or suspected pregnancy, suspected infection or sexually transmitted disease.

PRECAUTIONS:

- The use of oil-based contrast media such as ethyl esters may interact with the balloon of the catheter, causing possible balloon rupture. The use of oil-based contrast media is not recommended.
- Do not exceed the recommended balloon inflation volume of 1.5cc for the 5F H/S Catheter, or the balloon may burst.

ADVERSE EVENTS:

Some patients may have hypersensitivity to contrast media.

INSTRUCTIONS FOR USE:

- Open the over-wrap drape to form a 30" x 30" sterile field.
- Position the patient to allow pelvic access and prepare the vaginal area with the lubricant jelly and swabs provided.
- Follow the instructions included with the HIS Catheter provided in the tray to complete either the Hysterosonography or Hysterosalpingography procedure.
- The cervical dilator is used to find the external os by pushing the narrow end gently against the cervix until it enters the cervical canal. The shape retaining tip may be curved as necessary to facilitate insertion. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal os.
- The 18-gauge needle is provided to withdraw contrast media from its container using the 20 - 30ml syringe.
- The 36" extension tube set can be connected to the blue hub of the H/S Catheter and allows injection of the contrast media from outside the x-ray field during Hysterosalpingography procedures.

PRODUCT INFORMATION:

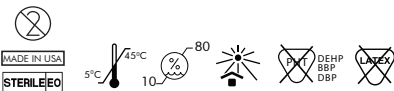
Product Number: TMI1198 / TMI1199
TMI1160 / TMI1161
TMI1157

Options: HSG Procedure Tray with
Flexible or Shapeable HSG Catheter 5F or 7F

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dba Thomas Medical, Inc.